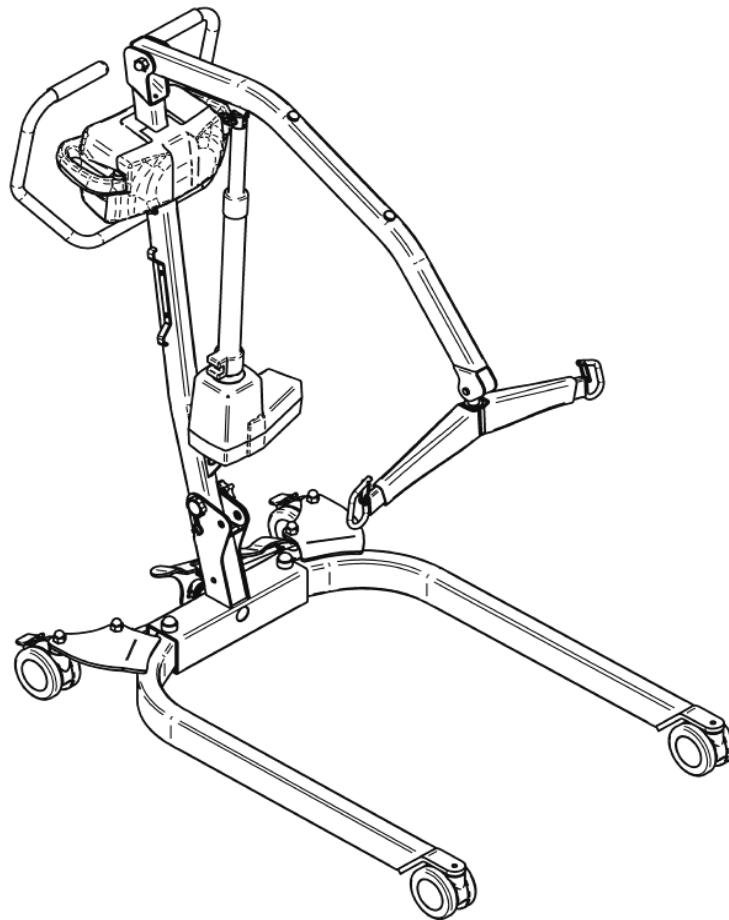
MODEL #  
82101041  
82101043  
82101044

# TECHNICAL MANUAL

## JUNIOR

### FOLDING LIFT

# JUNIOR

2001, Tanguay Street  
Magog (Quebec)  
Canada J1X 5Y5  
Phone: (819) 868-0441  
Toll free 1-800-868-0441  
Fax: (819) 868-2249  
E-mail: [www.bhm-medical.com](http://www.bhm-medical.com)

5155 Spectrum Way, unit 33  
Mississauga (Ontario)  
Canada L4W 5A1

14 Cross St.  
London, SW13 0PS, United Kingdom  
Tel: +44 (0)7740 717146 Fax: +44 (0)870 1619681

# TABLE OF CONTENTS

SYMBOLS .....	4
TECHNICAL DATA .....	5
SAFETY INSTRUCTIONS & WARNINGS .....	6
GENERAL .....	6
SHOCK PREVENTION .....	8
FIRE AND EXPLOSION .....	8
EQUIPMENT WARNING LABELS .....	8
ASSEMBLY INSTRUCTIONS.....	9
LIFT ASSEMBLY EXPLODED VIEW .....	13
LIFT PARTS LIST.....	14
BASE ASSEMBLY EXPLODED VIEW.....	15
BASE PARTS LIST.....	16
SPREADER BAR EXPLODED VIEW.....	17
CONTROL BOX DETAILS.....	19
CONTROL BOX PARTS NUMBERS.....	20
BATTERY PACK DETAILS .....	21
CHARGER WIRING DETAILS .....	22
BATTERY & CONTROL BOX WIRING DETAILS .....	23
BATTERY MAINTENANCE & SERVICE .....	24
MAINTENANCE & SERVICE.....	25
MONTHLY INSPECTION DETAILS.....	27
DIAGRAM.....	28
LOGBOOK .....	29
WARRANTY.....	32

COPYRIGHT 2003

**WARNING!**

All rights reserved. CONFIDENTIAL. The reproduction of this document or the transmittal in any form of any information, contained herein, without the authority in writing of an officer of the manufacturer is prohibited.

**TECHNICAL SPECIFICATIONS AND DESIGN SUBJECT TO CHANGE WITHOUT NOTICE.**

# SYMBOLS

**WARNING:** this symbol is intended to alert the user to hazards or unsafe practices which could result in serious bodily harm.

**CAUTION:** this symbol is intended to alert the user of the presence of important operating and maintenance instructions which could prevent product damage or possible personal injury.

**NOTE:** this symbol offers helpful information concerning certain operating procedures.

CONSULT ACCOMPANYING DOCUMENTS

BOOM "UP"

BOOM "DOWN"

INDICATES CHARGING MODE

INDICATES CHARGER POWER IS "ON"

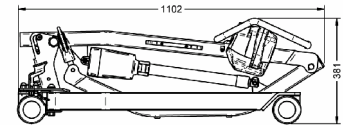
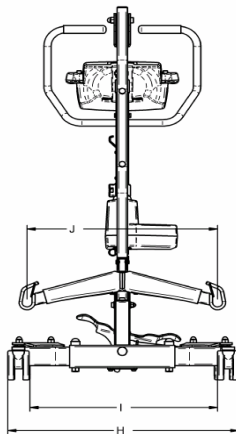
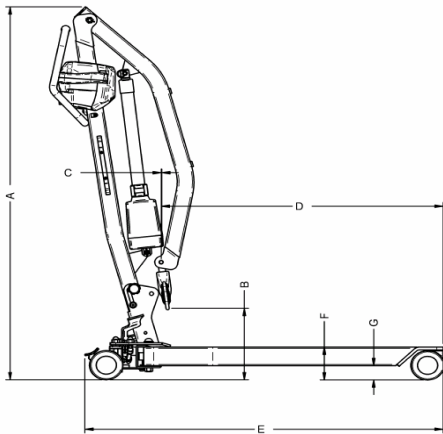
DO NOT ATTEMPT TO USE THIS EQUIPMENT WITHOUT UNDERSTANDING THIS MANUAL.

**To ensure safe operation, read carefully the entire manual, especially the section on "Safety Instructions and Warnings", before installing, operating or servicing this equipment.**

If anything is not completely understood, please contact your supplier for more details. Failure to comply with warnings in this manual may result in injury.

**Keep this manual with lift and refer to it as required. Contents of this manual are subjected to change without prior notice to users.**

# TECHNICAL DATA

FOLD

## DIMENSIONS

Description	Hoisting height at:					
	Min		Mid		Max.	
	mm	in	mm	in	mm	in
A Total lift height	1278	50.3	1245	49	1771	69.7
B Hoisting height at connecting points	246	9.69	1086	42.8	1618	63.7
C CSP Hoisting reach	3	0.12	631	24.8	452	17.8
D CSP from base end	851	33.5	223	8.78	400	15.7
	mm			in		
E Base total length	1090			42.9		
F Base height	112			4.4		
G Base floor clearance	48			1.9		
H Total external base width (legs closed)	700			27.6		
H Total external base width (legs open)	1110			43.7		
I Total internal base width (legs closed)	570			22.4		
I Total internal base width (legs open)	990			39.0		
J Connecting points distance	580			22.8		
Turning radius	945			37.2		
Minimum door requirement	660			26.0		

## WEIGHT DATA

Description	kg	lbs
Total lift	39.6	87.1
Shipping	45	87.1
Complete (base, mast with controls and actuator and boom with spreader bar)	35	77
Batteries pack	4.6	10.1

## TECHNICAL DATA

Description	Kg	lbs
Maximum Permissible Load (kg (lbs))	159	350
Maximum hoisting capability	199	438
IP rating control std. #700.160XX	IP21	
IP rating control opt. #700.161XX	IP24	
IP rating hand remote control	IP44	
Control voltage output (Vdc)	27.0Vdc	
Battery charger input (Vac)	90 to 240	
Battery charger output (VA max)	27	
Up & down current limiting (Amps.)	10A	
Fuse size (Amp.)	10A	
Duty Cycle 10%	6min/HR	
Sound power level ( dBA )	40	

ENVIRONMENTAL CONDITIONS: Normal operation: 10 C to 40 C, 0% to 90% R.H., 700-1060hPa.

Equipment is not suitable for use in the presence of flammable anaesthetic mixture with air or with nitrous oxide.

Equipment classification with respect to protection from electric shock: Internally powered.

Degree of Protection against Electric Shock: Type BF applied parts

# SAFETY INSTRUCTIONS & WARNINGS

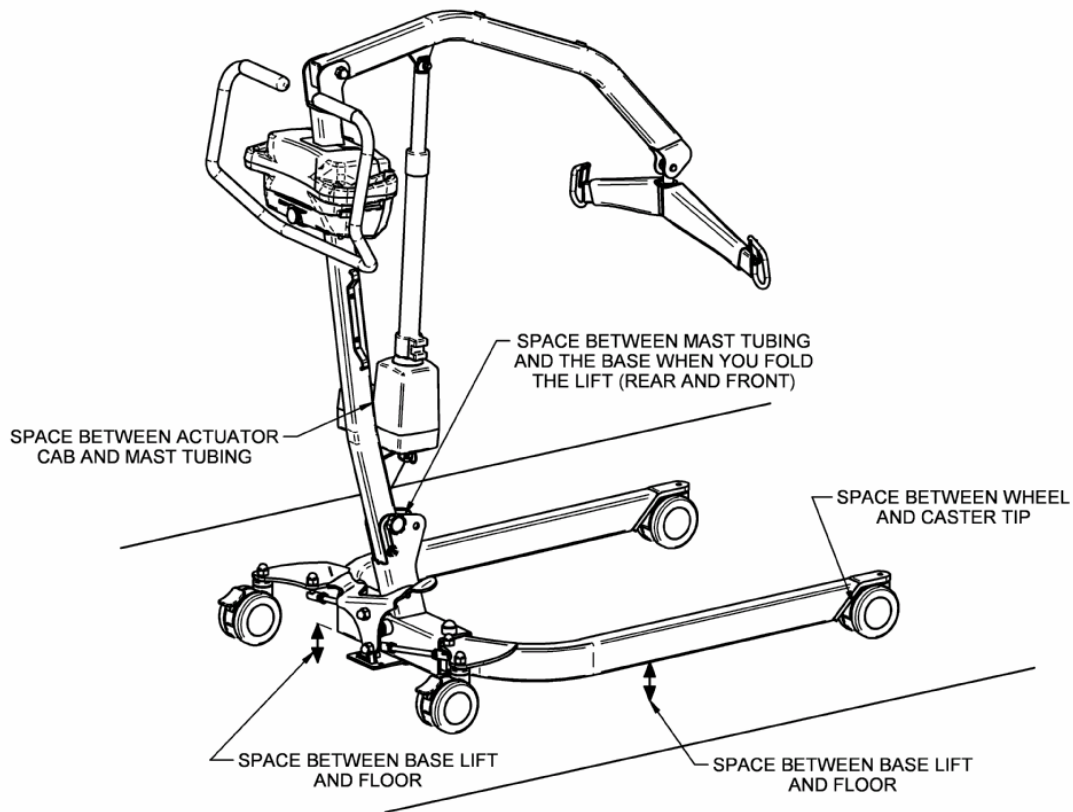
## GENERAL

- IMPORTANT – CAREFULLY READ THESE INSTRUCTIONS OR SERIOUS INJURY MAY RESULT.
- READ OPERATING MAINTENANCE INSTRUCTIONS IN THIS MANUAL BEFORE INSTALLING, OPERATING, OR SERVICING THIS EQUIPMENT.
- ALWAYS KEEP THESE INSTRUCTIONS WITH THE LIFT.
- YOUR LIFT is for transferring patients only. Do not use the lift for any other purpose.
- This mobile floor lift is not intended to be a transport device or to be used to transport a patient over any long distance.
- Do not attempt to manoeuvre the lift by pushing on the mast, motor shaft, boom or patient.
- DO NOT USE, in any way, the actuator as a handle to push or pull the lift. If the actuator is used as a handle, the caregiver and the patient safety is greatly compromised.
- ALWAYS manoeuvre the lift with the handles provided.
- DO NOT attempt to push or pull a loaded lift over a floor obstruction which the casters are unable to ride over easily.
- ALWAYS carry out the daily checklist before using the lift.
- IMPORTANT: Keep all components of the lift clean and dry, and have electrical and mechanical safety checkpoints done as instructed in the “Maintenance” section of this manual.
- Slings and accessories designed by any other manufacturer are prohibited and will void warranty. Use only the specified slings and accessories to maintain patient safety and product utility.
- DO NOT use a sling that is not recommended for the lift.
- NEVER use a damaged, torn or frayed sling.
- ALWAYS place the sling around patient according to the instructions enclosed.
- FOLLOW lifting procedures outlined in this manual.
- ALWAYS be prepared before attempting to transfer a patient.
- This mobile floor lift is intended to be used for patients within the specified weight limit indicated on the lift. Do not attempt to lift more than the weight limit indicated.
- BEFORE attempting to transfer, the patient must be assessed by a qualified professional.
- This mobile patient lift must be used by a trained caregiver to work with the patient to be transferred. This patient lift should never be used by a patient on their own.
- ONLY trained and qualified caregivers should transfer a patient. DO NOT attempt to use the lift if you have not been properly trained to do so.
- USE all controls and safety features only according to the rules specified in this manual. Never attempt to force a control or button on the lift.

Replace any precautionary or instruction labels that cannot be easily read.

## SAFETY INSTRUCTIONS & WARNINGS

- DO NOT store the lift in a shower, bath or other area with high humidity. DO NOT use an electric lift in a shower
- DO NOT push a loaded lift at speeds which exceed a slow walking pace (3 km/hour or 0.8 meter/second).
- DO NOT push the lift over uneven or rough ground, particularly if loaded.
- DO NOT BUMP the lift down steps, loaded or unloaded.
- DO NOT park a loaded lift on any sloping surface.
- BASE should not be opened as much as possible for optimum safety.
- Do not put fingers, hands or feet where space is limited (see diagram below). This could pinch, cut or seriously hurts any parts exposed.

# SAFETY INSTRUCTIONS & WARNINGS

## SHOCK PREVENTION

- AVOID violent shock during transportation.
- DO NOT touch or use a lift with bare conductors or a damaged power cord. Electrically live equipment can electrocute a patient. If the lift or charger has any exposed or damaged wires, contact your local dealer immediately.
- DO NOT splash or expose electric parts of the device to water or moisture.
- CHECK nameplate for voltage and cycle requirements. These requirements differ by country. Do not attempt to use the lift in an area that has a different voltage and cycle requirements.
- READ the battery and charger instructions thoroughly before using or storing them.

## FIRE AND EXPLOSION

- Batteries may explode, leak and cause personal injury if not disposed of properly.
- Do not place or store the battery under direct sunlight or near a heat source.
- Do not dispose of in fire or incinerate
- Do not short the battery terminals.
- Flush with water if electrolyte (Acid) comes in contact with skin or eyes.
- Batteries must be recycled or disposed of according to local law regulations. When returning batteries, insulate their terminals with adhesive tape otherwise the residual electricity in use batteries may cause fire or explosion.

### IMPORTANT :

BATTERIES NEED TO BE CHARGED FOR A MINIMUM OF 8 HOURS PRIOR TO USING THIS LIFT FOR THE FIRST TIME.

## EQUIPMENT WARNING LABELS

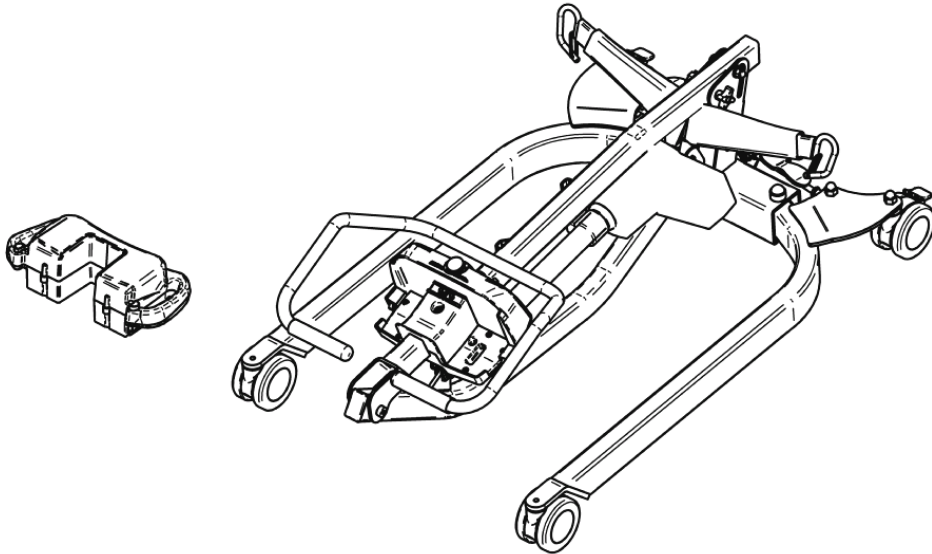
- Inspect all precautionary labels of the equipment. Order and replace all labels that cannot be easily read.

# ASSEMBLY INSTRUCTIONS


## Assembly Instructions for Junior Lift

1. Remove all parts from carton and inner boxes, make sure all components are present (see checklist on the main box sticker).

Check any for any shipping damage. Do not discard carton or packing material until you are completely sure there is no damage to the lift.

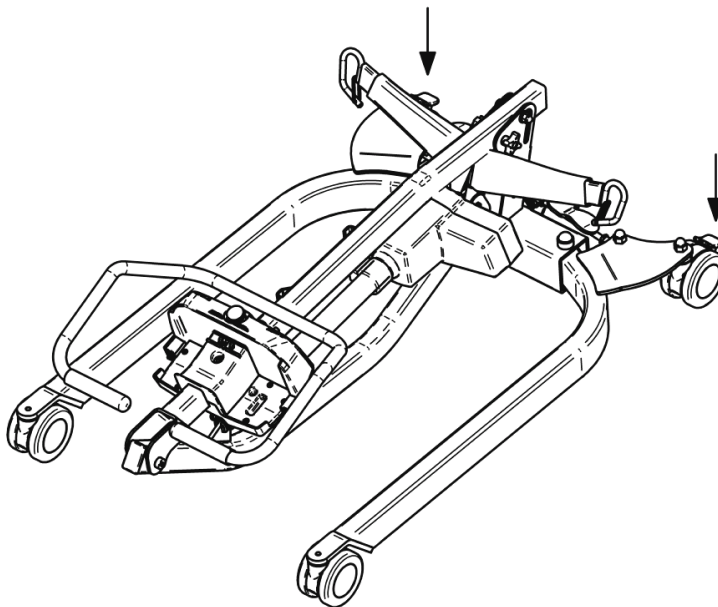
2. Place base assembly on the floor and lock both rear casters.


## ASSEMBLY INSTRUCTIONS

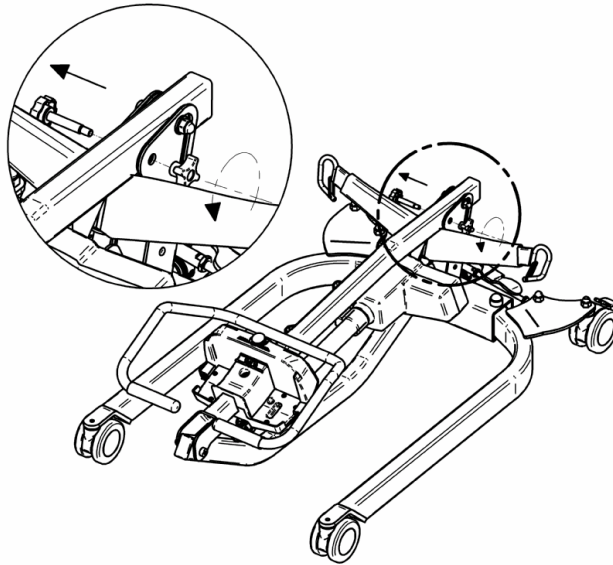3. Remove the shoulder knob and the turning knob from the mast.

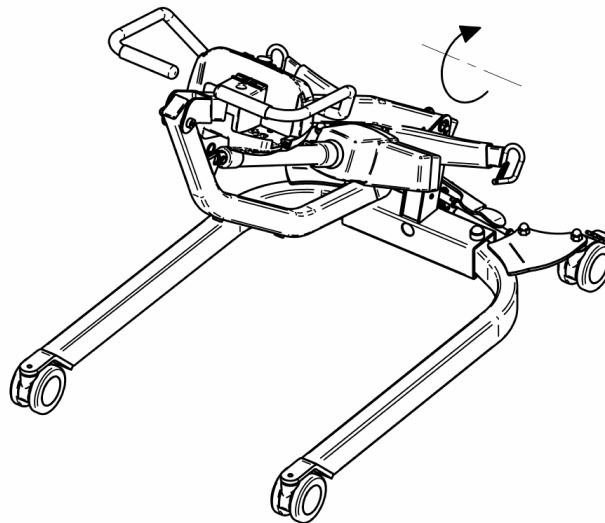4. Unfold the lift until it is straightened out.

## ASSEMBLY INSTRUCTIONS

5. Insert the shoulder knob back to block the mast (Fig. A), then tighten the turning knob (Fig. B). Unstrap the spreader bar from the mast. Insert the straps inside the spreader bar padding and zip it.

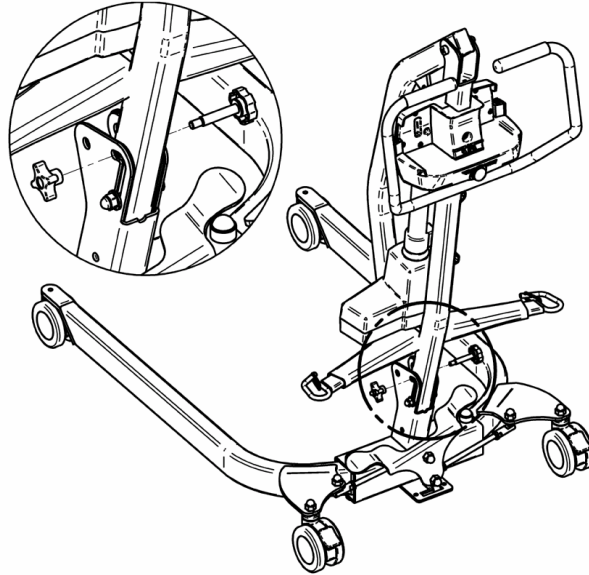
Fig. A



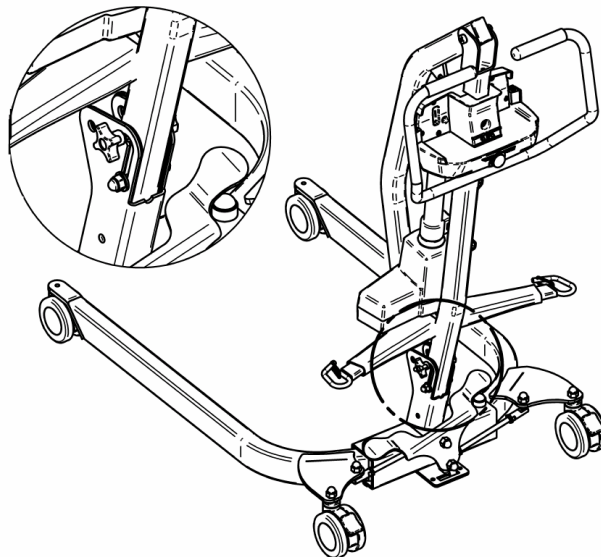
Fig. B

## ASSEMBLY INSTRUCTIONS

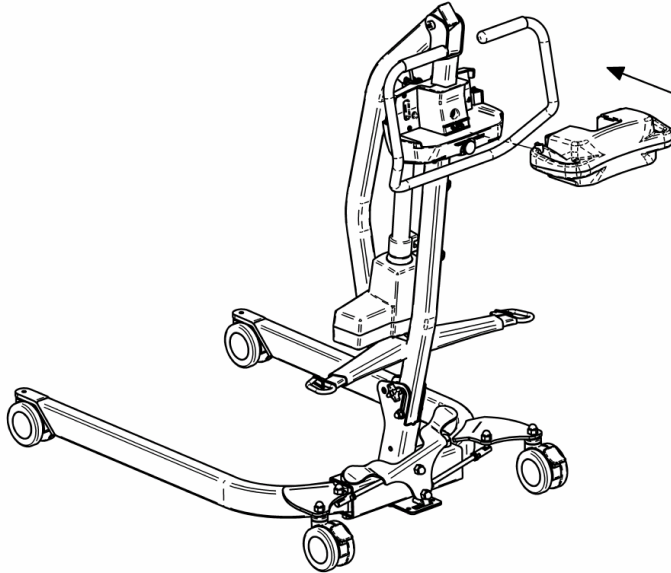
6. Place battery into the control box.

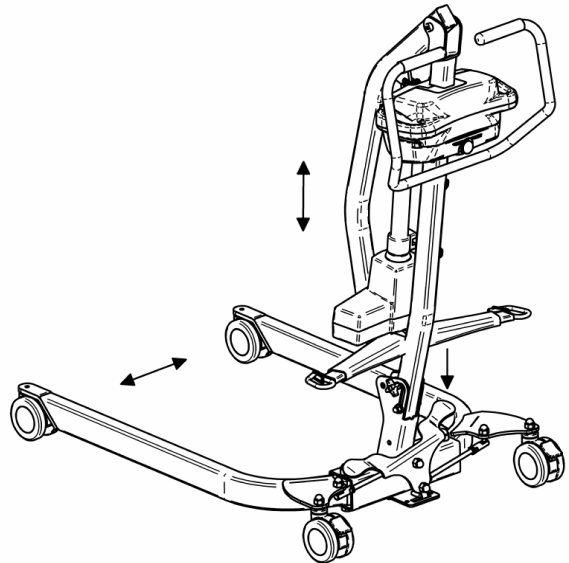7. Test for proper leg spread by disengaging the pedals, then, push it down from left to right. Test for proper **HI/LO** operation of the boom – Run boom to both its highest and lowest points. The lift is fitted with limit stops; it should cease to function when the boom reaches its highest or lowest travel.

# LIFT ASSEMBLY EXPLODED VIEW

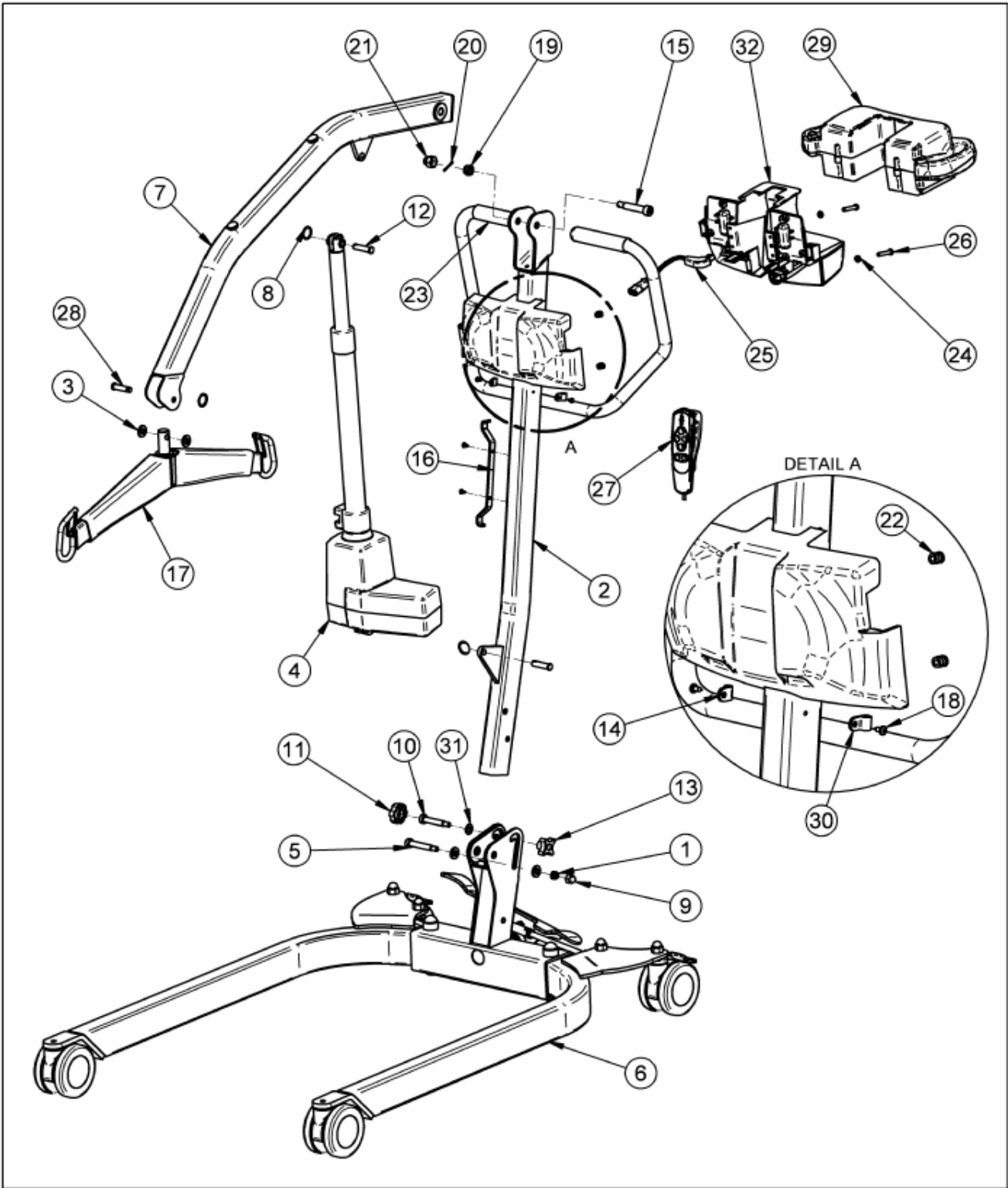
## LIFT PARTS LIST

Item Number	Document Number	Title	Qty
1	000.03800	LOCKNUT M8 ZINC	1
2	222.23116	MAST ASSY	1
3	000.04450	NYLON FLAT WASHER 10 X 2.5T	4
4	E6520	LA34-350 ACTUATOR	1
5	000.04285	SHOULDER 10 X 50 (M8)	1
6	700.23113	BASE KIT ASSY	1
7	700.23121	BOOM KIT ASSY	1
8	000.00921	SPLIT RING	3
9	000.03815	BLACK PLASTIC CAP M8	1
10	000.04284	SHOULDER 10 X 45 (M8)	1
11	000.04440	KNOB M10	1
12	200.06150	CLEVIS 10 x 38.5	2
13	000.03965	TURNING KNOB M8	1
14	E0295	CABLE CLAMP 3/16"	1
15	000.04605	SHOULDER 12 X 50 (M10)	1
16	222.20800	POWER CORD BRACKET	1
17	700.05496	SPREADER BAR ASSY	1
18	000.00635	TAP SCREW ST4.2 X 9.5 TORX	4
19	000.04410	LOCKNUT M10 ZINC	1
20	000.01800	COTTER PIN 2 X 20 ZINC	1
21	000.04422	BLACK PLASTIC CAP M10	1
22	000.03405	NUT INSERT AT M6 X 1.0	2
23	M5715	BLACK RUBBER HANDLE 7/8"	2
24	000.03420	LOCKWASHER M6 ZINC	2
25*	402.00027	CTRL BOX CLASS 2 SPT CORDSET 90	1
26	000.03140	M6 X 35 BUTTON ZINC	2
→ 27	700.13610	FLOORLIFT 2 BUTTONS HANDSET	1
28	000.00903	CLEVIS 3/8" X 1 5/16"	1
→ 29	700.16200	BATTEY PACK ASSY	1
30	E0296	CABLE CLAMP 5/16"	1
31	000.05110	FLAT WASHER 0.406 X 0.75 X 0.060 ZINC	1
32	(SEE PAGE 19)	CONTROL BOX ASSY	1

\*Only required with a control box with integrated charger, except those with UL cord.

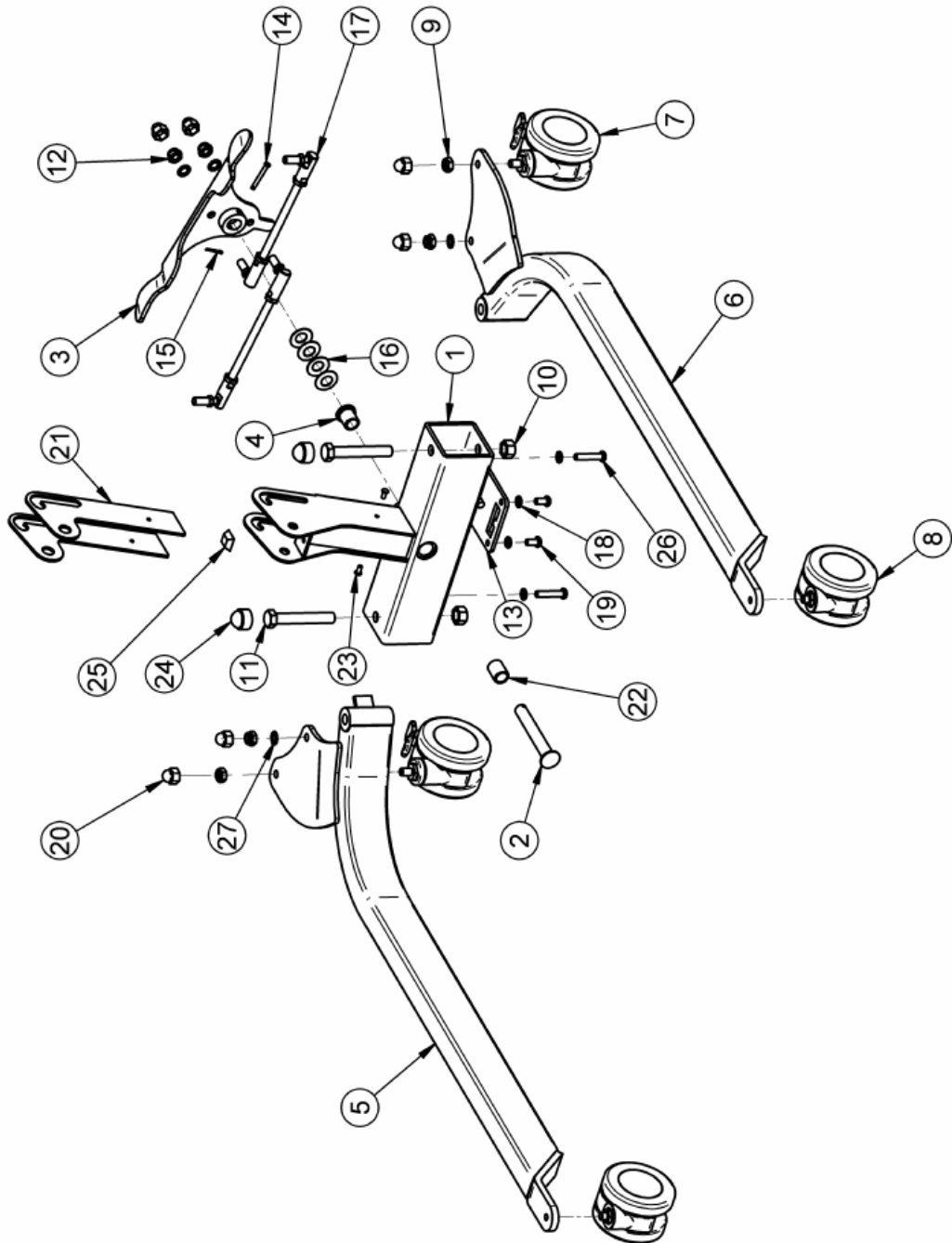
# BASE ASSEMBLY EXPLODED VIEW # 700.23113

## BASE PARTS LIST # 700.23113


<b>#700.23113 (BASE KIT ASSEMBLY)</b>			
<b>Item Number</b>	<b>Document Number</b>	<b>Title</b>	<b>Qty.</b>
1	222.23113	BASE ASSY	1
2	222.23114	PIVOT SHAFT	1
3	222.23132	PEDAL ASSY	1
4	000.05460	FLANGE BUSHING 5/8"ID X 13/16"OD	1
5	222.23001	RIGHT LEG ASSY	1
6	222.23006	LEFT LEG ASSY	1
7	000.20435	4" SWIVEL WHEEL WITH BRAKE	2
8	000.20430	4" SWIVEL WHEEL	2
9	000.04411	LOCKNUT FUJILOK M10 ZINC	2
10	000.05010	LOCKNUT STOVER M14 ZINC	2
11	000.05000	SCREW M14 X 90 HEX ZINC	2
12	000.04410	LOCKNUT M10 ZINC	4
13	222.20095	BASE SHIFTER PLATE	1
14	000.00898	CLEVIS 3/16" X 1 5/8"	1
15	000.00860	COTTER PIN 3/32 X 3/4 ZINC	1
16	000.04920	SPRING WASHER 1 1/4"OD X 0.63"ID	4
17	000.20120	TIE ROD END KIT (L-R)	2
18	000.03960	LOCKWASHER M8 ZINC	2
19	000.03613	SCREW M8 X 16 BUTTON ZINC	2
20	000.04422	BLACK PLASTIC CAP M10	6
21	200.23112	JUNIOR SLIDE PLATE	2
22	000.05462	BUSHING 5/8"ID X 13/16"OD	1
23	000.00895	ALUM RIVET 3/16" 1/8-1/4	2
24	000.05015	BLACK PLASTIC CAP M14	2
25	000.20188	BLACK BUMPON	2
26	000.03690	SCREW M8 X 40 BUTTON ZINC	2
27	000.04426	STAR WASHER M10 ZINC	4
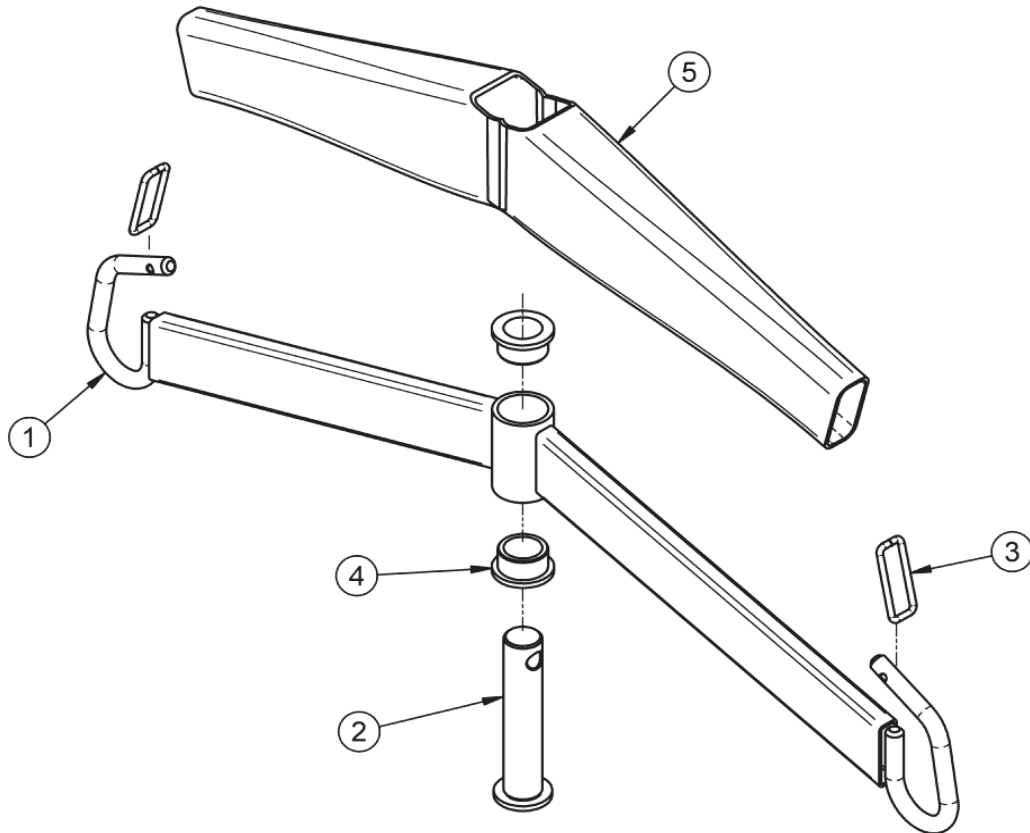
# SPREADER BAR EXPLODED VIEW # 700.05496


<b>#700.05496 (SPREADER BAR ASSEMBLY)</b>			
Item Number	Document Number	Title	Qty.
1	222.05455	SPREADER FRAME	1
2	200.20480	PIVOT SHAFT	1
3	200.05466	SAFETY LATCH	2
4	000.05465	FLANGE BUSHING 20MM	2
5	361.05496	PADDING SPREADER BAR	1

# BOOM ASSEMBLY EXPLODED VIEW # 700.23121
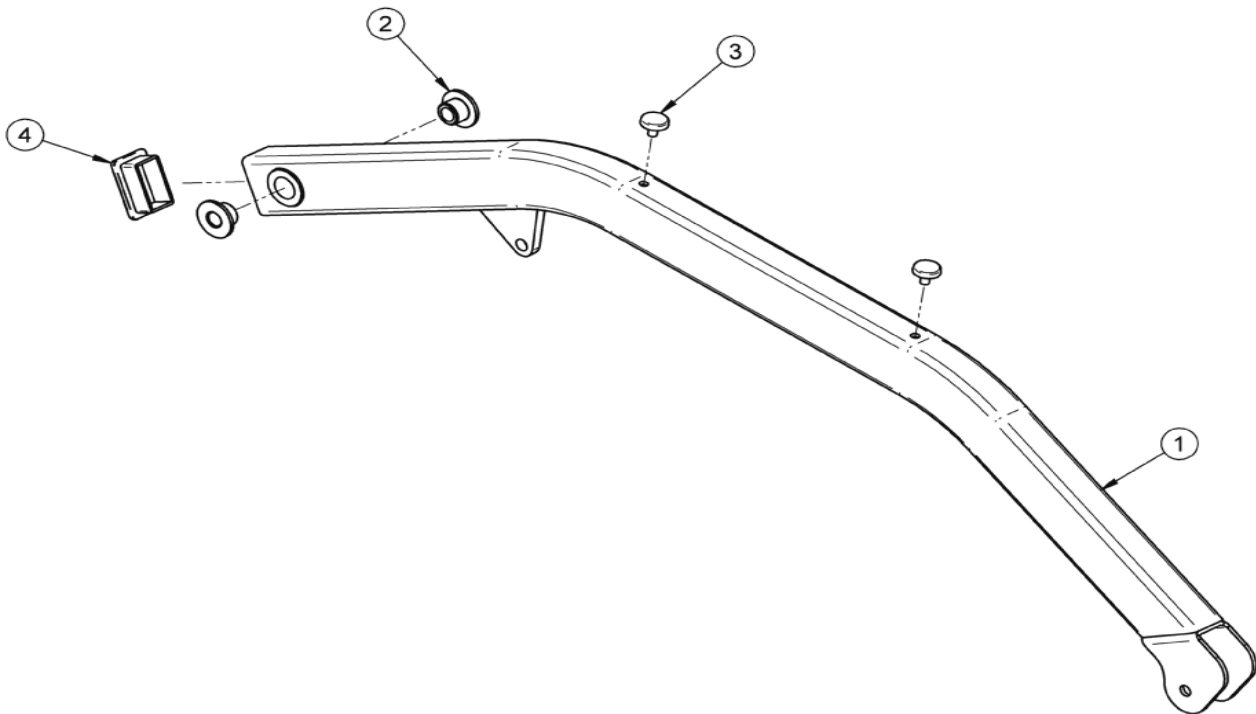
# 700.23121 (BOOM ASSEMBLY KIT)			
Item Number	Document Number	Title	Qty.
1	222.23121	BOOM ASSY	1
2	200.20250	BOOM BUSHING	2
3	000.20186	FLAT HEAD SPLIT BUMPER	2
4	000.20146	TUBE INSERT 1 1/4" X 2"	1
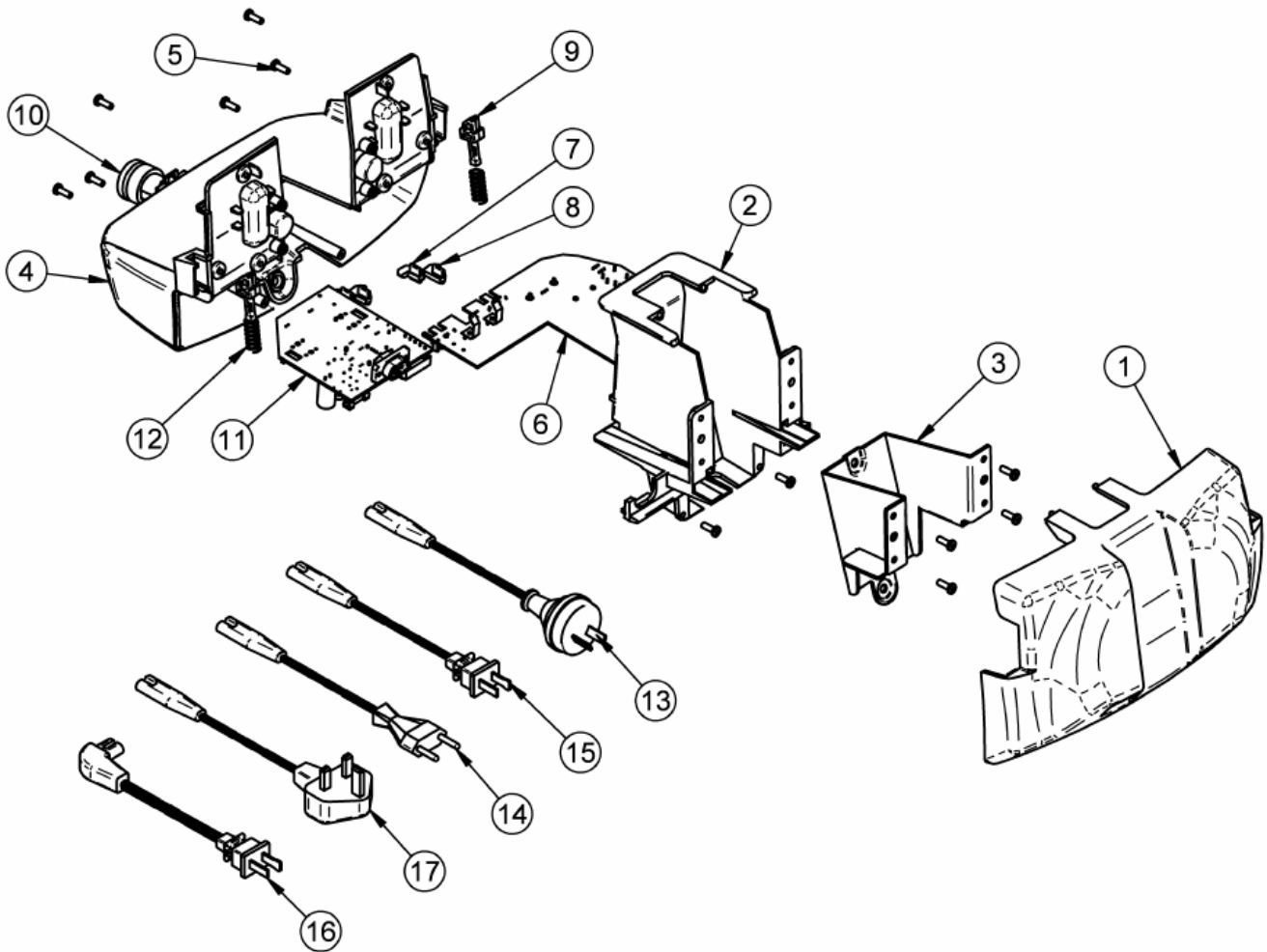
# CONTROL BOX DETAILS (With integrated charger) # 700.16008.2


# CONTROL BOX PARTS NUMBERS

<b>#700.16008.2 (CTRL BOX WITH INTEGRATED CHARGER)</b>			<b>#700.16008.2</b>
<b>Item Number</b>	<b>Part Number</b>	<b>Title</b>	<b>Qty</b>
1	250.16053	CTRL BOX BACK	1
2	250.16103	CONTROL BOX MIDDLE	1
3	200.16260	CONTROL BOX STEEL BRACKET	1
4	200.16005	CONTROL BOX FRONT "ABDE"	1
5	000.02480	TAP SCREW PLASTITE 4.0 X 12	12
6	492.00045-3	CTRL BOX CIRCUIT (CAL. 10A)	1
7	200.16240	CONTROL BOX RIGHT BUTTON	1
8	200.16230	CONTROL BOX LEFT BUTTON	1
9	200.16250	CONTROL BOX LATCH	2
10	C6P22MM	PUSH BUTTON	1
11	492.00031-4	CHARGER CIRCUIT	1
12	000.00881	COMPRESSION SPRING	2

## POWER CORD (DEPENDING OF THE COUNTRY)



<b>Item Number</b>	<b>Part Number</b>	<b>Title</b>	<b>Qty</b>
13	402.00004	CLASS 2 AUSTRALIA SPT CORDSET	1
14	402.00003	CLASS 2 CE SPT CORDSET	1
15	402.00002	CLASS 2 JAPAN SPT CORDSET	1
16	402.00013	CLASS 2 UL SJT CORDSET 90	1
17	402.00005	CLASS 2 UK SPT CORDSET	1
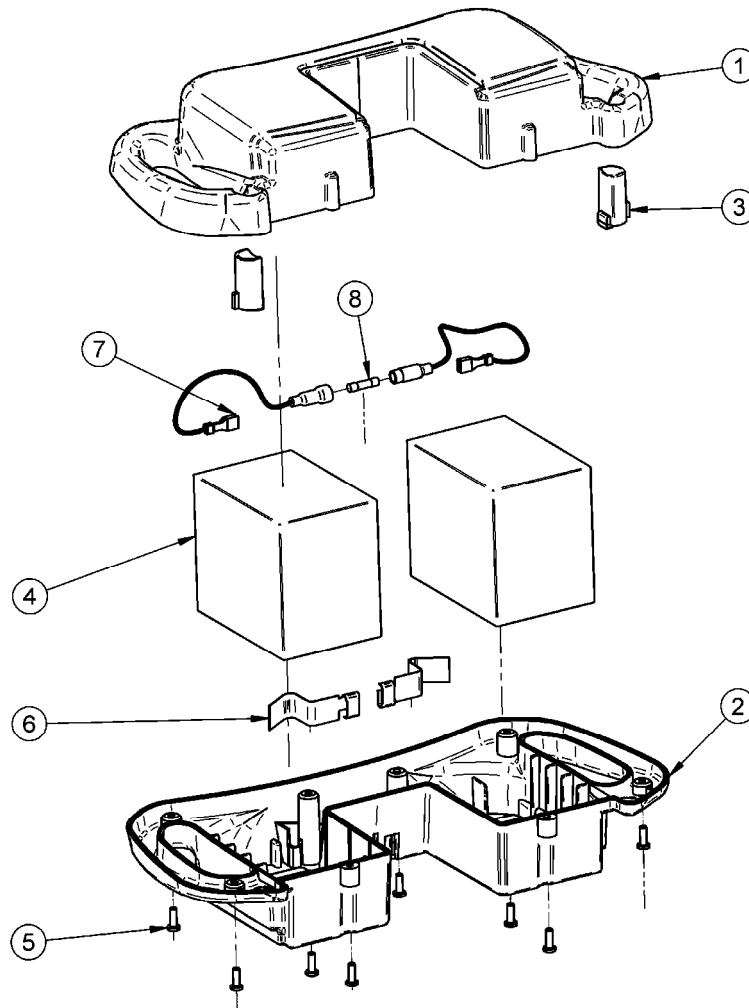
# BATTERY PACK DETAILS


Item Number	Document Number	Title	Qty
1	200.16200	BATTERY PACK TOP	1
2	200.16210	BATTERY PACK BOTTOM	1
3	200.16220	BATT PACK RELEASE BUTTON	2
4	403.10500	BATTERIE 12V 5Ah	2
5	000.02480	TAP SCREW PLASTITE 4.0 X 12	8
6	200.16270	CONTACT BATTERY PACK	2
7	402.00033	WIRE JUMPER BATTERY PACK	1
8	C8FGMA10	FUSE GMA	1

# CHARGER WIRING DETAILS
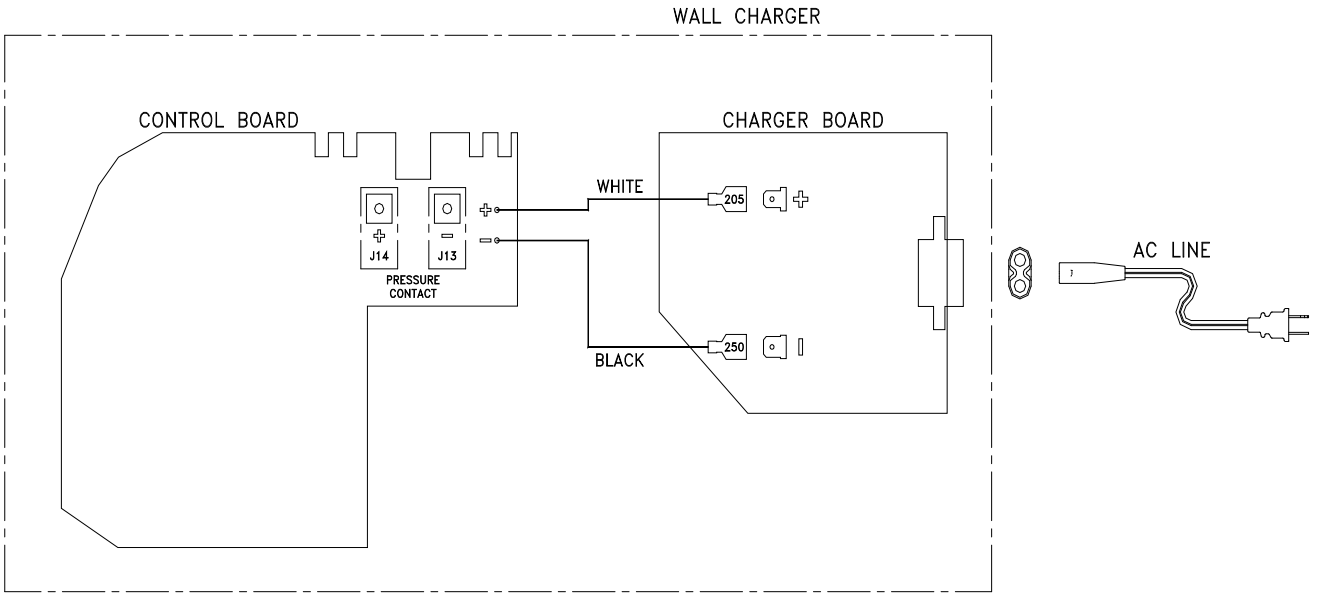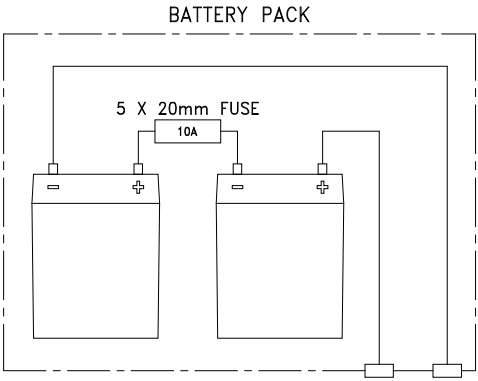
# BATTERY & CONTROL BOX WIRING DETAILS (with integrated charger)
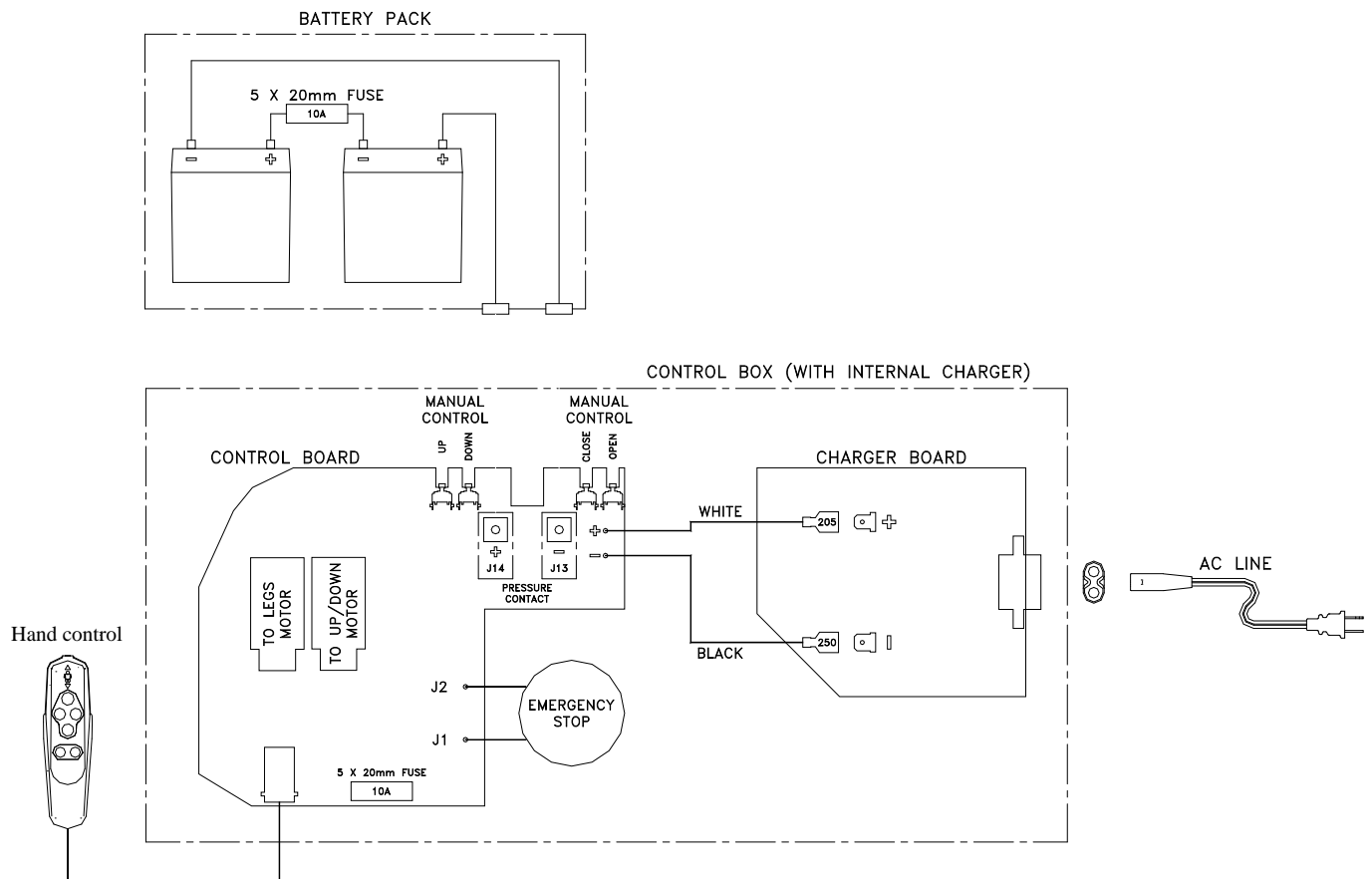
# BATTERY MAINTENANCE & SERVICE

## MAINTENANCE AND REPLACEMENT OF BATTERIES

### **Battery Systems:**

The Junior uses sealed lead-acid batteries mounted within the control box. Unlike nickel-cadmium batteries, lead-acid batteries are not subject to a memory effect. Therefore they should not be completely drained before recharging. The control box is equipped with an automatic control shut off after 20 seconds of inactivity to prevent any battery drainage while the lift is in the stand-by mode. Power to the lift is reactivated by pushing any handset button.

### **IMPORTANT :**

CONNECT THE CHARGER AT LEAST 8 HOURS BEFORE THE FIRST USE OF THE LIFT IN ORDER TO COMPLETELY RELOAD THE BATTERIES.

**Do not drop the battery. A dropped battery may have been internally damaged and left unable to hold a charge. The green LED on the charger may not accurately reflect the charge condition of a damaged battery.**

**Never attempt to use a battery that appears to be damaged or dented.**

**Do not store the battery near a heat source.**

**Always keep the metal battery contacts clean.**

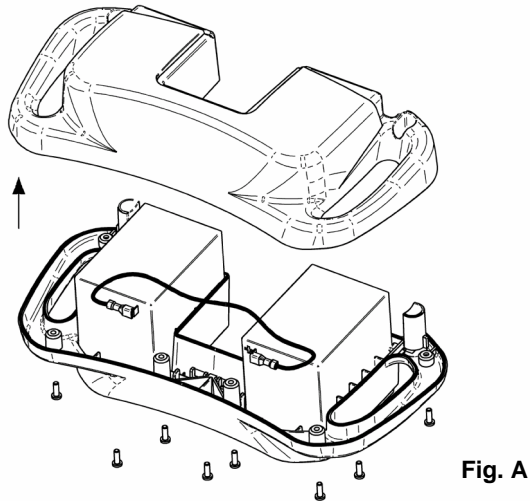
### **Maintenance:**

- For optimum battery lifetime, the control box/battery pack must be connected to the main voltage as often as possible.
- The batteries must be charged at least every 2 weeks.
- The batteries should be replaced at least every 4 years. The pattern of use, frequent and high-powered discharges will reduce the battery life.
- The battery pack and control box should have a full function test at least once a year.



# MAINTENANCE & SERVICE

## MAINTENANCE AND REPLACEMENT OF BATTERIES

To remove the battery pack from the control box; at the same time push the 2 latch buttons located on each side of the control box and pull straight out toward you. The battery pack shell is formed of 2 pieces. Remove the 8 screws and with the latch buttons facing up, lift off the top shell. (Fig. A)

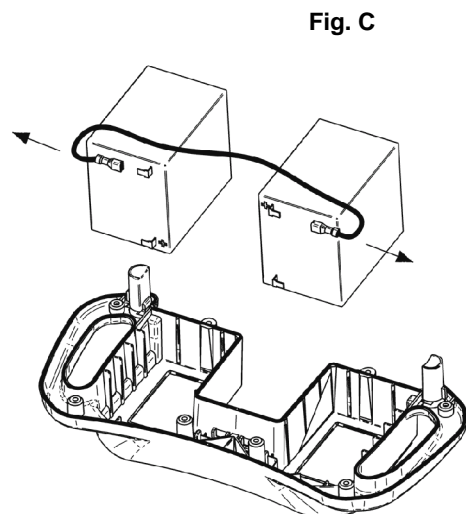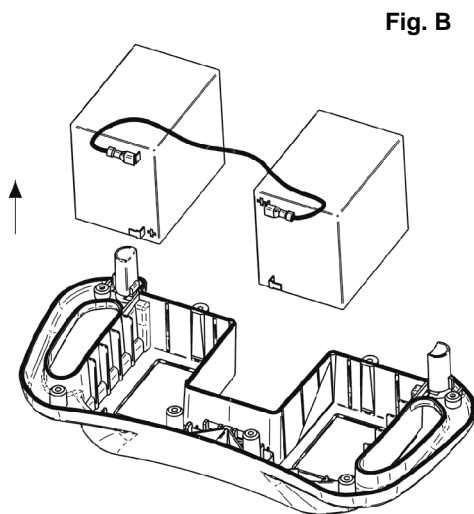Lift out the 2 batteries and remove the fused jumper from the connectors. (Fig B & C)  
**Warning: Make sure to not short batteries.**

Check the following:

- Is the fuse OK, if not replace it with an identical 10 amp fuse. Part # C8FGMA10.
- Are the wire and battery connectors free from corrosion?
- Is the cable damaged, if so replace it with Part (see exploded view)?
- Is there any damage or evidence of abuse to the battery pack casing? A complete battery pack assembly can be ordered (See exploded view)

To reassemble the Battery pack:

- Insert the batteries into the bottom shell with the battery terminal end first. Make sure there is a solid contact between the terminals and the packs steel connector.
- With both batteries completely seat in the bottom casing, install the fused jumper between each top terminal.
- Replace the top shell and reinstall the 8 screws.

## MONTHLY INSPECTION DETAILS


**Refer to diagram on the following page for the location of each inspection area.**

These procedures are supplied as a guideline for personnel who are responsible for the maintenance and safety inspections of patient handling devices. The manufacturer accepts no responsibility for ongoing inspections and checks. These are guidelines only and are not intended as a substitute for a regular, thorough equipment maintenance program.

**Note:** if during your inspection any bolt, nut or structural fastener is removed, it is imperative that you reinstall all associated safety devices such as cotter pins, split rings and lock nuts. If one of these devices is defective, you must replace it with an original new component.

ITEM #	PROCEDURE	√
1	ENSURE THE PIVOT CLEVIS PIN IS WELL SECURED BY THE SPLIT RING AT BOTH ENDS OF THE ACTUATOR	
2	ENSURE THAT THE PIVOT SHOULDER BOLT IS SECURELY FASTENED AND THE COTTER PIN IS IN PLACE	
3	ENSURE THAT THE CLEVIS PIN IS PROPERLY INSERTED INTO THE SPREADER BAR BLOCK AND SECURED BY THE SPLIT RING.	
4	CHECK FRONT AND REAR CASTERS REGULARLY FOR HAIR AND DEBRIS, CLEAN AS NECESSARY, CHECK BRAKE FUNCTION ON REAR CASTERS.	
5	INSPECT ALL WELD SITES FOR CRACKING OR SEPARATION.	
6	ENSURE ALL THE NUTS ARE SECURE ON THE STUDS, ENSURE JAM NUTS ARE SECURE ON SHAFTS	
7	CHECK TO ENSURE THAT THE LOCKNUT IS TIGHT.	
8	CHECK THAT THE BOLT IS SECURE.	
9	RUN ACTUATOR FROM LIMIT TO LIMIT TO ENSURE LIMIT STOPS ARE FUNCTIONING.	
10	PRESS BUTTON TO ENSURE THAT ALL ELECTRICAL POWER IS CUT OFF.	
11	CHECK ALL THE FUNCTIONS ON THE HAND CONTROL – CHECK TO ENSURE WIRES ARE NOT PULLED OUT OF CONNECTOR ON HAND CONTROL CASE.	
12	CHECK FOR THE SMOOTH FUNCTION OF THE UP/DOWN SWITCHES.	
13	CHECK BATTERY PACKS FOR DAMAGE – CHECK THAT THEY WILL ACCEPT A CHARGE BY PUTTING THEM IN THE CHARGER.	
14	CHECK THE FUNCTION OF THE CHARGER LIGHTS (ON THE LIFT OR ON WALL CHARGER) BY INSERTING A BATTERY PACK – ENSURE THAT POWER LIGHT IS FUNCTIONAL.	
15	CHECK THAT THE SAFETY LATCHES ARE IN PLACE AND FUNCTIONAL.	
16	CHECK THE FUNCTION OF THE EMERGENCY LOWERING DEVICE BY APPLYING WEIGHT TO LIFT AND ACTIVATING THE LOWERING DEVICE.	
17	MAKE SURE THAT THE CUTTER PIN OF THE PEDAL IS IN ITS PLACE. THIS WILL INSURE THE STABILITY AND THE SAFETY OF THE BASE AND PEDAL	
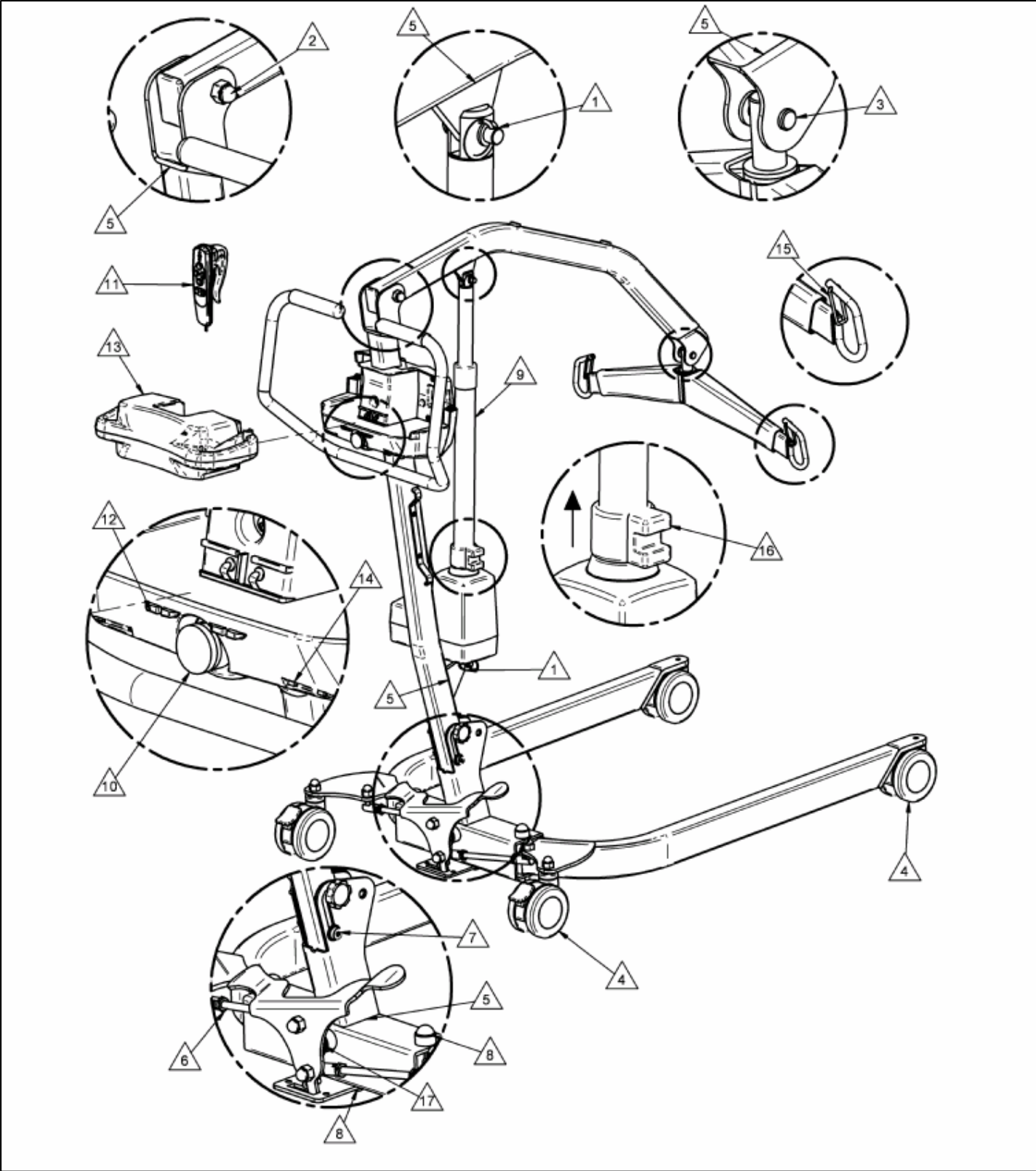
# DIAGRAM

# LOGBOOK

Product Name : JUNIOR

Product Serial Number :

Date of purchase :

Warranty Period :

Distributed By :

Owner Checklist :

- Ensure the lift is serviced regularly as the Monthly inspection details on page 22.
- Contact an authorised Distributor / Service Agent immediately if there are any problems with the operation of the device.
- Ensure the logbook is completed and signed.
- Record any repairs required.
- Withdraw the patient lifter from service if inspection reveals that user safety is jeopardised in any way from use of the patient lifter.

## TO BE COMPLETED AFTER EACH SERVICE OR INSPECTION

Service Type:	Periodic inspection	Minor	Major
Condition report :	_____ _____		
Action taken :	_____ _____		
Date :	Inspected by:		
_____	_____		_____
	Printed	Signature	

Service Type:		Periodic inspection	Minor	Major
Condition report :				
_____				
_____				
Action taken :				
_____				
_____				
Date :		Inspected by:		
_____		_____		
		Printed		Signature

Service Type:		Periodic inspection	Minor	Major
Condition report :				
_____				
_____				
Action taken :				
_____				
_____				
Date :		Inspected by:		
_____		_____		
		Printed		Signature

Service Type:		Periodic inspection	Minor	Major
Condition report :				
_____				
_____				
Action taken :				
_____				
_____				
Date :		Inspected by:		
_____		_____		
		Printed		Signature

Service Type		Periodic inspection	Minor	Major
Condition report :				
_____				
_____				
Action taken :				
_____				
_____				
Date :		Inspected by:		
_____		_____		
		Printed		Signature

# WARRANTY

This warranty is extended only to the original purchaser/user of BHM products.

BHM Medical Inc. warrants its products to be free from defects in material under normal use and service, within the periods stated below from the date of purchase. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced at BHM Medical option. This warranty does not include any labour or shipping charges incurred in replacement part installation or repair of any such product. BHM Medical sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

Patient Lifter	1 year
Weighing Devices	1 year
Accessories on Lifter	1 year
Slings	1 year
Batteries - All other lifts	1 year

For warranty service, please contact the dealer from whom you purchased the BHM Medical product. [In the event that you do not receive satisfactory warranty service, please contact BHM Medical (see contact information in on the cover page).]

Do not return products to our factory without prior authorization. BHM Medical will issue a Return Merchandise Authorization (RMA) Number. C.O.D. shipments will be refused; all shipments to BHM Medical must be prepaid. For this warranty to be valid, the purchaser must present its original proof of purchase at the moment of the claim. The defective unit, assembly or part must be returned to BHM Medical for inspection. The part or components repaired or replaced are guaranteed for the remaining period of the initial warranty.

## **Limitations and Exclusions:**

The warranty above does not apply to serial numbered products if the serial number has been removed or defaced.

No warranty claim shall apply where the product or any other part thereof has been altered, varied, modified, or damaged; either accidentally or through improper or negligent use and storage. Warranty does not apply to products modified without BHM Medical express written consent (including but not limited to products modified with unauthorized parts or attachments), products damaged by reason of repairs made to any component without the specific consent of BHM Medical, or to products damaged by circumstances beyond BHM Medical control. BHM Medical will solely determine evaluation of warranty claim. The warranty does not apply to problems arising from normal wear or failure to adhere to the instructions in this manual. BHM Medical Inc. slings are void of warranty if not laundered as per instructions on the Sling Label.

BHM Medical Inc. shall not be liable for damages losses or inconveniences caused by a carrier.

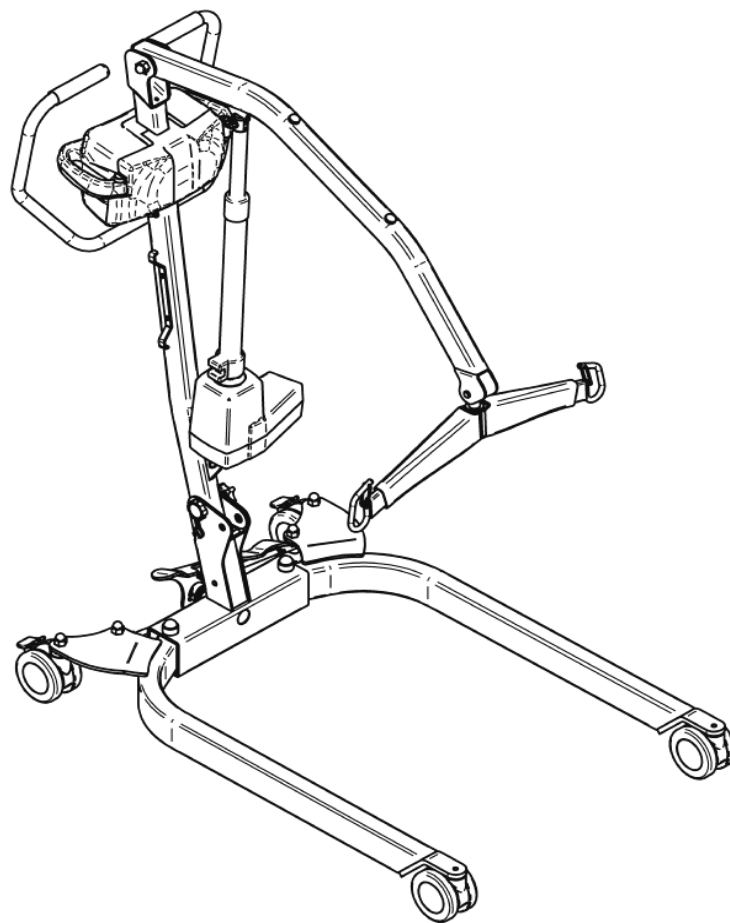
This warranty replaces any other warranty expressed or implicit and constitutes BHM Medical Inc. only obligation towards the purchaser. BHM Medical shall not be liable for any consequential or incidental damages whatsoever.

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